


ORALMEDIC[®]

Instructions for use

Barrier Solution

Oral Ulcer Pain Relief. The one-time, 5-10 second topical treatment for Canker and Mouth Sore Pain.

INDICATIONS: ORALMEDIC[®] is indicated in the topical treatment of ulcerating lesions of the oral cavity such as Canker and Mouth Sores (Oral Ulcers). ORALMEDIC[®] provides fast permanent relief from the pain and discomfort of oral mucosal ulcers. ORALMEDIC[®] is not intended for the treatment of Oral Blisters.

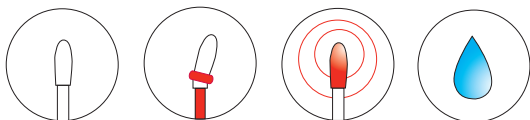

INSTRUCTIONS FOR USE

1. Dry the affected area using the drying swab.
2. Hold applicator with colored ring up. Bend the colored ring to one side until it snaps. ORALMEDIC[®] will flow to the opposite end.
3. Apply wet swab to the dried affected area holding swab in contact for 5 - 10 seconds. A brief "stinging" sensation is experienced immediately upon application.
4. Rinse mouth thoroughly with water and spit out.

Do not get on clothing, as it may discolor or damage fabric.
Dispose in trash..

NOTE: One application per ulcer treatment is usually sufficient. However, if the ulcer pain returns shortly after rinsing with water, it is an indication that some part of the ulcer was not covered. Repeat application one more time following the directions above. It is not recommended that more than one treatment session be performed on an individual oral ulcer.

WARNINGS: Keep out of the reach of children. Do not use if allergic to sulphur in any form. If eye exposure occurs, immediately remove any contact lenses and rinse thoroughly with water.

PRECAUTIONS: General - ORALMEDIC[®] is for topical use on oral cavity mucosa only. Avoid eye contact. Safety and effectiveness in pregnant women and children under the age of 12 has not been established.

ADVERSE REACTIONS: ORALMEDIC[®] may cause local irritation upon administration. If excess irritation occurs during use, rinse thoroughly. If condition persists contact a health care professional.

HOW SUPPLIED: Swab Applicator Contains 0.20 ml of product. Store at room temperature, 15° C – 30° C or 59° F – 86° F.

CONTAINS: HybenX[™] (Hydroxybenzenesulfonic Acid, Hydroxymethoxybenzenesulfonic Acid, Sulfuric Acid, Water, FD&C Red #40). ORALMEDIC[®] has been developed using HybenX[™] technology.

MANUFACTURED BY: EPIEN Medical, Inc.
1-888-884-4675 ■ www.epien.com